

The Genio[®] system

Implantable Stimulator

A small stimulator is implanted under the chin close to tongue muscle nerves



Activation Chip and Disposable Patch

The activation chip connects to a disposable patch and contains a battery and the customized stimulation program



Charging Unit

The charging unit is used to charge the battery of the activation chip



Doctor Contact Information

Name

Phone number

Hospital

GENIO

Indication for Use: Genio[®] is intended to treat adult patients suffering from moderate to severe Obstructive Sleep Apnea (OSA) and who have either not tolerated, have failed or refused Positive Airway Pressure (PAP) therapy.

All rights reserved © 2020 Nyxoah S.A. All content on this brochure, including the texts, trademarks, service marks, logos, illustrations, photos, graphics, design, etc., are the property of Nyxoah S.A. Nyxoah S.A. owns all rights with respect to any of their trademarks, service marks, logos, and copyrights appearing on this brochure.

Patented and design protected technology.
Doc. ID: MS-GEN-002078-EN Rev02

CE marked since 2019.

Caution – Investigational device in the United States. Limited by United States federal law to investigational use.

genio.nyxoah.com

Patient information

Restful. Nights.

**CAUTION -
Investigational device.
Limited by Federal law
to investigational use.**

GENIO[®]

What is Genio®

Genio® is intended to treat patients suffering from moderate to severe Obstructive Sleep Apnea (OSA).

Genio® stimulates tongue muscle nerves, contracting the tongue and keeping the airway open. This prevents sleep events from occurring, such as apneas and hypopneas, allowing the user to enjoy a restful night.


The first intention of the Genio® is not to treat snoring but it might have a positive impact on it.

If you are interested in this therapy, consult your doctor.

Be aware that

- No alcohol consumption is permitted 24 hours prior to sleep studies
- Male users must shave chin area on a daily basis

Daily Use Process

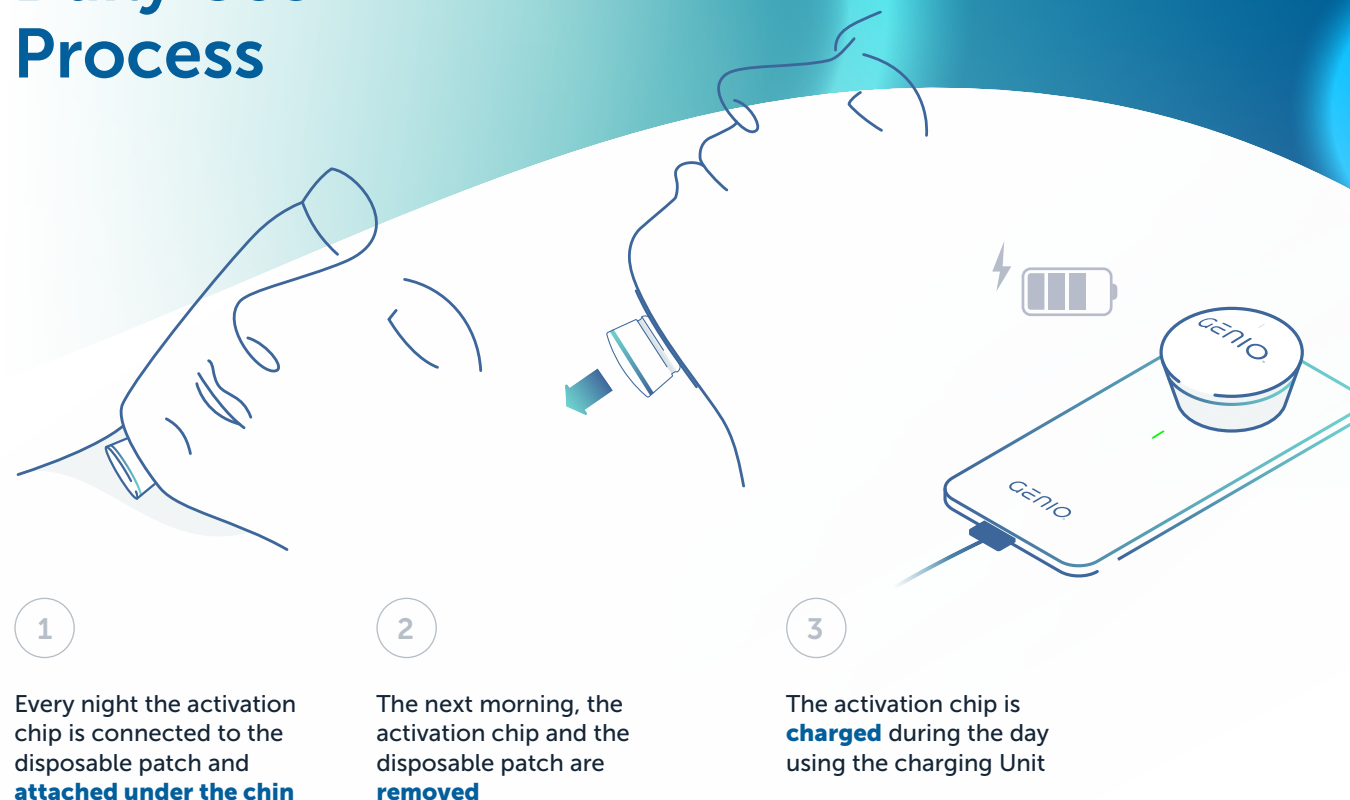

Recovery / Healing 6 - 8 weeks

Implantation of the stimulator

- You will undergo a short and minimally invasive procedure under general anesthesia
- The implant will be placed through a **small incision** under your chin

First device activation

- You will receive **training** on your new Genio® system
- You will feel the **stimulation** for the first time
- **Sensitivity and intensity** of this new experience will differ from person to person and change over time
- It might take time to **adjust** to this sensation and specific tongue movement

Therapy Personalization

- You will need at least one **sleep study** so your doctor can adjust your therapy settings. Your doctor may need to schedule additional sleep studies to monitor and adjust your therapy settings.
- In order to optimize and reach therapeutic levels, it is very important to **use the system each night** and to attend **followup appointments** scheduled by your doctor.

CAUTION - Investigational device.
Limited by Federal law to investigational use.